

# Action+ Bandage

## Final Bandage System



AB1004M Shown

**HCPCS CODE**  
**#A4463**

***Holds sub-bandages  
and medical devices without  
adhesives - Non-absorbent - Reusable***

**Contents: 1 Medium White Sleeve**  
**Item No. AB1002M - 12.5cm x 4.5cm x .3cm**  
**Approximate range of comfort: 18cm to 40cm circumference**

# Action+ Bandage

- No Adhesives
- No Skin Tears
- No Bruising
- No Latex
- No Seams
- No Fasteners
- No Discomfort
- No Waste



Action Bandage is the first bandage system that is cost effective, non-absorbent, easy to apply, and reusable. Action Bandage holds dressings, sub-bandages and medical devices in place without the use of sticky tapes, messy adhesives or clumsy fasteners. Soft, cool and comfortable.

### **Easy to apply:**

- Clean and dress affected area
- Hold Action Bandage vertically and open the top end with both hands
- Roll the bandage downward creating a doughnut shape
- Stretch the doughnut-shaped bandage from the center and place near affected area
- Unroll the bandage over the affected area
- Adjust the bandage so the air holes are equal in shape and size

*Clean and redress wound according to Physician instructions – Do not apply Action Bandage directly to the wound – Remove if skin irritation, swelling or pain occur and consult a Physician - For added comfort place a non-adherent cotton layer between skin and Action Bandage.*

### **Easy to reuse:**

To clean and refresh the Action Bandage, wash with warm water and antibacterial soap. Rinse thoroughly, remove moisture with a clean towel and air dry completely. Place bandage in plastic bag, sprinkle with antifungal or talcum powder and shake until bandage is lightly covered. The Action Bandage is ready to use again! Non-powdered use dampen and dry to remove powder.

*When Action Bandage becomes dirty or sticky, repeat this procedure.*

Made in China • Patent Pending • For more info visit: [www.actionbandage.com](http://www.actionbandage.com)  
**888.900.5988** • 61741 Campground Rd. Washington, MI 48094

**Size: 12.5 x 4.5cm x .3cm - Approximate range of comfort: 18cm to 40cm circumference**  
**Medium White Sleeve - Item No. AB1002M**